## Southern Nevada Public Health Laboratory TEST MENU

## **Hepatitis B Surface Antibody (HBsAb)**

Method	Chemiluminescent Microparticle Immunoassay
CPT Code	80074 Organ or Disease Oriented Panels
	86317 Qualitative or Semiquantitative Immunoassays
Specimen	Seum
Collect in	Serum Separator Tube (SST)
L.L.P.	I de la constant de l
Labeling	Label transport tube with patient's first and last name or coded identification number. Record date and time collected on the tube.
	date and time concered on the tabe.
Dunanaian	Do not according to the Contribution of the Co
Processing	Do not open transport tubes. Centrifuged specimens are placed directly on an instrument or stored in the refrigerator until next business day.
	stored in the renigerator until next business day.
Transport	Serum Separator Tube (SST)
·	
Sample Rejection	Quantity Not Sufficient for testing. <300 uL
Requisition	SNPHL Request Forms
	Testing is performed only at the request of SNHD
Turn Around Time	48-72 hours following receipt in our laboratory
	10 72 means removing receipt in our laboratory
Results	Non-Reactive, borderline, Reactive
Reported	System generated fax; Electronic transmission
•	
Note	Immunocompromised patients may not react positive with this test, and present as a false
	negative.
Contact	Southern Nevada Public Health Laboratory – Clinical Laboratory Section