

Southern Nevada Public Health Laboratory
TEST MENU

Hepatitis Panel, Acute (HCV, HBsAg, HBc IgM, HAV IgM)

Method	Chemiluminescent Microparticle Immunoassay
CPT Code	80074 Organ or Disease Oriented Panels
Specimen	Seum
Collect in	Serum Separator Tube (SST)
Labeling	Label transport tube with patient's first and last name or coded identification number. Record date and time collected on the tube.
Processing	Do not open transport tubes. Centrifuged specimens are placed directly on an instrument or stored in the refrigerator until next business day.
Transport	Serum Separator Tube (SST)
Sample Rejection	Quantity Not Sufficient for testing. <300 uL
Requisition	SNPHL Request Forms Testing is performed only at the request of SNHD
Turn Around Time	48-72 hours following receipt in our laboratory
Results	Non-Reactive, borderline, Reactive
Reported	System generated fax; Electronic transmission
Note	Immunocompromised patients may not react positive with this test, and present as a false negative.
Contact	Southern Nevada Public Health Laboratory – Clinical Laboratory Section